 Let's Protect™

---

# ACTISORB® SILVER 220

ACTIVATED CHARCOAL DRESSING WITH SILVER

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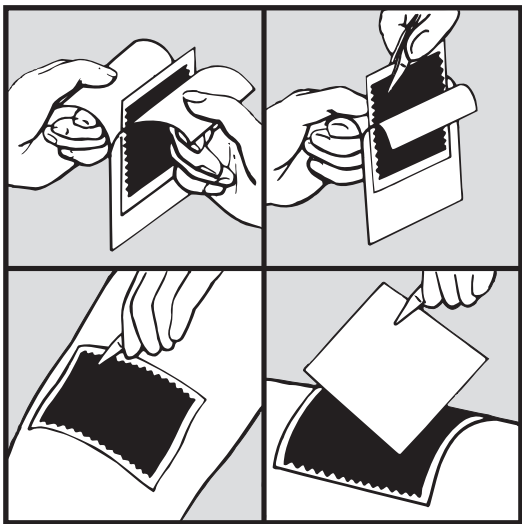
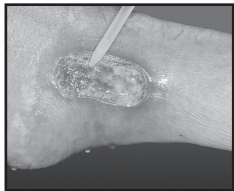
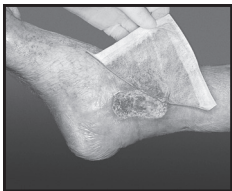
6.

## **EN** Product Description

ACTISORB® Silver 220 Activated Charcoal Dressing with Silver is a dressing composed of pure activated carbon impregnated with silver ( $33 \mu\text{g}/\text{cm}^2$ ). The dressing creates a favourable environment for effective wound healing by binding and immobilising micro-organisms which contaminate and infect wounds. The silver impregnation combats wound micro-organisms in the dressing, which reduces the bacterial colonisation of the wound and inhibits infection. ACTISORB® Silver 220 dressing is tolerated very well and because of its purely physical wound cleansing principle there is no fear of side effects or the development of resistance such as can occur when using antibiotics.

By eliminating offensive odours ACTISORB® Silver 220 dressing improves the quality of life for the patient and nursing personnel.

### **Indications**

ACTISORB® Silver 220 dressing is suitable as the first therapeutic step in the management of all chronic wounds. It is indicated for fungating carcinomas, ulcerative, traumatic and surgical wounds where bacterial contamination, infection or odour occurs.

### **Precautions**

Clinicians / Healthcare Professionals should be aware that there are very limited data on prolonged and repeated use of silver containing dressings, particularly in children and neonates.

### **DIRECTIONS FOR USE**

#### **Preparation of wound site**

Dry, necrotic tissue must first be removed by surgical, enzymatic or autolytic debridement.

#### **Application**

- Prior to application, ACTISORB® Silver 220 dressing can be impregnated with saline solution or sterile water.
- The adsorptive properties of the dressing are most fully utilised when ACTISORB® Silver 220 dressing makes direct contact with the wound, (see photo 2).
- Either side of ACTISORB® Silver 220 dressing can be placed next to

the wound site.

- ACTISORB® Silver 220 dressing can be easily packed into deep wounds, (see photo 3).
- The outer nylon surface enables removal from the majority of wounds without adherence or trauma.
- If required, ACTISORB® Silver 220 dressing can be placed on top of a non-adherent wound contact layer, (see photo 4). In this case it is important to ensure that the mechanism of action of ACTISORB® Silver 220 dressing is not impaired by excess ointment or grease of the contact layer.
- Depending on the level of exudate an absorbent, secondary dressing may be placed on top of ACTISORB® Silver 220 dressing, (see photo 5).
- Fix the dressing in place in a manner appropriate for the indication.
- Naturally, work according to a sterile procedure.

### **Changing the dressing**

ACTISORB® Silver 220 dressing can remain in situ up to 7 days, dependent on the level of exudate, while the secondary absorbent dressing is changed as required. Initially it may be necessary to change ACTISORB® Silver 220 dressing every 24 hours, (see photo 6).

N.B.: ACTISORB® Silver 220 dressing should not be cut, otherwise particles of activated carbon may get into the wound and cause discoloration.

**Do not re-use.**

**Do not re-sterilise.**

**Do not use if package is damaged.**

**The use by date of this product is printed on the packaging.**

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©2018 Systagenix Wound Management Limited,